# USER MANUAL

Hereby, MOB, declares that item KC6422 is in Compliance with the essential requirements and other relevant conditions of the medical device directive 93/42/EEC (MDD). The full text of the EU declaration of conformity is available at the following internet address: [www.mobmanual.com](http://www.mobmanual.com).

Distributed in the EU by.  
**MOB, PO BOX 644, 6710 BP (NL).**

Manufacturer:

HANGZHOU FOREIGN ER & TRADE SERVICE CO., LTD.  
58 Chang Sheng Road, 310006 Hangzhou CN

PO: 41-XXXXX  
Made in China

KC6422

**EN**

First aid kit:

**Includes:**

**Medical Device Directive (MDD):**

3x Long adhesive bandages.  
2x Round adhesive bandages  
2x Bandage (5cm)  
1x Bandage (3.5cm)

- The adhesive bandages are sterile unless the package is damaged or opened
- For single use only
- Store in a dry place

Clean and dry the wound and wound environment. Remove the plasters from the packaging and remove the backing paper without touching the wound pad. Place the wound pad over the wound and press the edges well. Check the wound regularly and change the patch at least once a day. Do not use on wounds with a lot of exudates. In case of skin irritation, remove the patch and do not continue to use the product. Consult a doctor if healing is not forthcoming or irritations persist.

**General Safety Regulation (GS):**

1 pair of scissors.  
3x cotton ball- wrap in a paper Ø1.8x6cm.  
1x adhesive tape.  
6x cotton swabs.

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